

DISPOSABLE TOURNIQUET

QUICK RELEASE BUTTON

INSTRUCTIONS FOR USE

Application

1. **Position the Tourniquet:** Hold one end in each hand, place the tourniquet under the patient's arm, and bring both ends around to the front.
2. **Support the Button:** To avoid over-tightening or damaging the skin, support the button with two fingers while applying.
3. **Create Tension:** Keep the button end of the tourniquet close to the arm. Pull the other end upwards, then over the button to create tension.
4. **Secure the Tourniquet:** Gently press the button end of the tourniquet through the hole to lock it in place.

Removal

1. **Prepare for Release:** Place your index and middle fingers underneath either side of the excess tourniquet and slide them down to the button.
2. **Release the Button:** Use your thumb to hold down the button, rotate your hand 90° upwards, and gently push the button back through the hole to release.

APPLICATION



REMOVAL

