
### INSTRUCTIONS FOR USE

Inflate

Maintain Pressure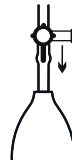

Deflate

#### EN

##### INTENDED USE

- The Ezy-Aid Pressure Infusion Bags are Single Patient Use bags with an inflation bulb intended to pressurise IV Bags to increase flow rate of fluid delivery to patients.

##### CAUTIONS

- For trained professional use.
- Prior to use, prime the administration set to remove residual air.
- Avoid exposure to freezing or excessive heat conditions.
- Bag pressure should not exceed 300mmHg.
- Pressure gauge is for indication only and will not provide precise measurement of internal pressure.
- Check bag for damage and inflate completely to ensure no leaks.

##### INSTRUCTIONS FOR USE

- Write patient name on bag.
  - Inserts fluid bag, between mesh and air bladder panel.
  - Using integrated hook, hang the pressure infusion bag and fluid bag on I.V pole.
  - Turn valve to 'Inflate' position as shown on bag illustration and using inflation bulb, inflate the bag to desired pressure.
  - Turn valve to 'Maintain Pressure' position. Check bag pressure at regular intervals.
  - Turn valve to 'Deflate' position as shown on bag illustration to release pressure.
  - This bag is for Single Patient Use and should be cleaned and air-dried between uses. The bag may be reused upto a maximum of 20 times on the same patient.
- Always follow local clinical protocols to ensure the product is used in compliance with hospital procedures, infection control guidelines and risk assessment.

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